Fractora:

VERSATILE SKIN RESURFACING & REJUVENATION

DESIGNER DERMIS APPROACH TO FULL SCALE TREATMENT ALL IN ONE APPLICATOR



Designer Dermis:

Fractora can deliver a customized dermal remodeling, or "Designer Dermis", by the selection of the tip depth and density that best addresses the vertical layer of the dermis.

> • Superficial: Skin tone, texture, photo damage, fine lines

• Deep Dermal Remodeling: Wrinkles, scars, vascularities

| 10-15 mj/pin | Non Ablative |
|--------------|--|
| 20-30 mj/pin | Ablative Less Coagulation (Fractional Erbium effects) |
| 40-62 mj/pin | Ablative (Fractional CO2 effects) |
| 50 mj/pin | Vessel Ablation |




| NER DERMIS APPROACH TO FULL SCALE TREATMENT ALL IN ONE APPLICATOR | | | | | |
|---|--|--|--|--|--|
| | | | | | |
| | | | | | |
| 60 | 60 Pin 00 microns | 126 Pin 600 microns | 24 Pin 3000 microns | 24 Pin - Coated 3000 microns | |
| Superficial: | Yes | ` Yes | Yes | No | |
| Density: | Moderate | High | Low | Low | |
| Deep RF: | Moderate | No | Yes | Yes | |
| th | thorse' tip leading to e regeneration of | Closely grouped dense columns of ablation. | Consistent re-epithelialization throughout the entire | Deep thermal stimulation and contraction, ideal for | |
| col | llagen and thicker dermis. | | epidermis and deep dermis. | ethnic skin and deep wrinkles. | |
| | | | | Positivo / Nogativo | |

Out from the Dark Ages ...

Traditionally CO2 lasers produced miraculous results, but with weeks of healing equal to that of a burn victim followed by prolonged redness, pain and possible hyper or hypo-pigmentation. Fractional lasers avoided many of these issues, but with limited outcomes and multiple sessions required.

Fractora bridges the gap. Fractora provides profound contraction, moderate and deep wrinkle results without the weeping and oozing, prolonged flushing and loss of pigment. In addition, areas like the neck that had previously been unaccounted for can now be safely and effectively treated.

With Fractora, the fractional RF leaves the epidermal and dermal tissue intact between the columns of ablation to serve as a reservoir for re-epithelialization and dermal remodeling. Additionally, the RF energy heats the skin in a controlled manner that allows a multi-layered impact with volumetric bulk heating, while leaving the skin's surface intact.

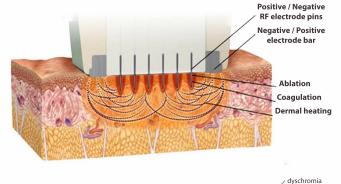









Combined effects

Re-epithelialization and Phagocytosis

Remodeling

FRACTORA[™] 1.855.411.2639 www.inmodemd.com